

Jacoti Lola™ 1.0 Quick Start Guide



What does Jacoti Lola™ do?

Jacoti Lola™ is a solution which enables remote wireless transport and enhancement of audio in difficult hearing situations in the classroom. It transmits enhanced audio over Wi-Fi from the speaker to the listeners so as to improve the listener's speech understanding in noisy or not acoustically well-suited environments.

Jacoti Lola™ comprises an iOS application dedicated to listeners and a Mac OS X application called Jacoti Lola Hub, which broadcasts the speaker's voice and/or the computer's system audio to all the connected listeners with negligible delay.

Additionally, Jacoti Lola™ allows a listener to broadcast his/her own voice to his/her peer listeners.

Installation

In order to get Lola running, two applications need to be installed:

- Jacoti Lola Hub on your Mac computer: Jacoti Lola Hub is available on the Mac App Store for free for up to 2 clients.
- Jacoti Lola on your iOS device/s; Jacoti Lola is available on the iOS App Store.




Once you have both applications installed, please also make sure that they are connected to the same Wi-Fi network.


Getting started

Once Jacoti Lola Hub is installed on your Mac computer:

1. Double-click the Jacoti Lola Hub icon that you can find in your Applications folder
2. The Jacoti Lola Hub icon will show up in your computer status bar. By default it automatically starts a session that shares your microphone audio and multimedia content.
3. You can open the Jacoti Lola Hub main panel at any time by clicking on its icon in the status bar.

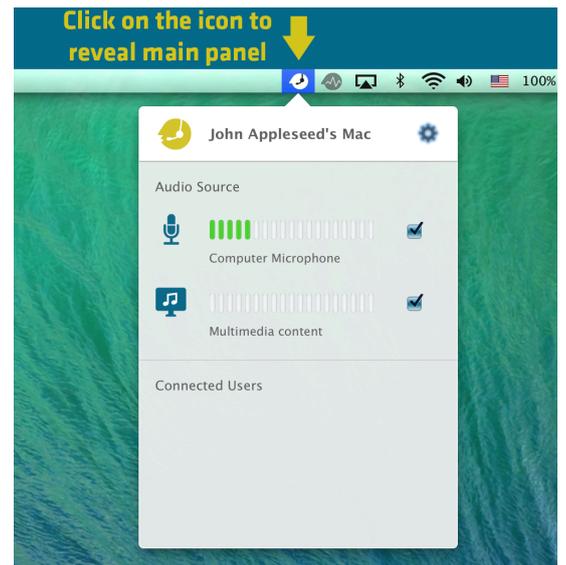

Then, on your iOS device:

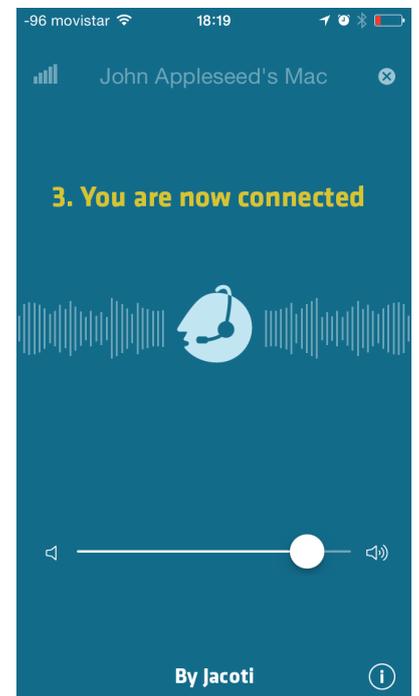

1. Execute the app and plug your earphones.
2. A session should appear on the list, with the name of the computer you have Jacoti Lola Hub running on.
3. You should now be connected and listening to what the host has to say